Dualex™, Dualex™-Plus, and Dualex™-Ultra Transducer Protectors

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Made in Ireland
Merck Millipore Ltd.
Tullagreen,
Carrigtwohill,
Co. Cork, IRL

© 2012 EMD Millipore Corporation. All rights reserved.
The M logo, Dualex, and Durapel are trademarks of Merck KGaA, Darmstadt, Germany.
Millipore is a registered trademark of Merck KGaA.
Luer-Lok is a trademark of Becton, Dickinson and Co.
PR04010, Rev. A, 02/12

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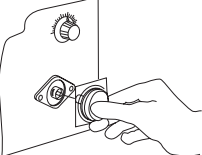
Hergestellt in Irland

English

Dualex™ Transducer Protector
Cat. No. SLGVS25XS (0.2 µm, 50/box)

Dualex™-Plus Transducer Protector
Cat. No. SLGVS25PS (0.2 µm, 50/box)

Dualex™-Ultra Transducer Protector
Cat. No. SLGVS25US (0.2 µm, 50/box)

■ Single use only ■ Sterile ■ Non-pyrogenic

Introduction

This insert describes how to use the Dualex™, Dualex™-Plus, and Dualex™-Ultra Transducer Protectors. Each transducer protector consists of a 0.2 micron (µm) Durapel™ filter sealed in a plastic housing. The product is used as an in-line protective device for pressure monitors in hemodialysis systems. It protects patients from bacteriological and particulate cross-contamination. It also prevents blood and dialyzing fluids from damaging sensitive transducer monitoring devices. In addition, the Dualex™-Plus and Dualex™-Ultra devices are equipped with Luer-Lok™ connectors, making them ideal for use in high-flux hemodialysis systems.

Directions for Use

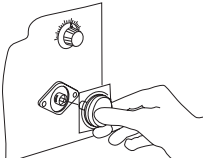
WARNINGS:

- To ensure sterility, do not use this product if the package is damaged.

CAUTIONS:

- Do not resterilize or reuse the transducer protector, as Merck Millipore Ltd. cannot assure the sterility, integrity, and performance beyond a single use.
- Do not use the transducer protector at temperatures above 45 °C (113 °F).

- Remove the cover from the Dualex™ Transducer Protector using aseptic techniques.
- Attach the transducer protector to the appropriate male Luer connection on the hemodialysis system and remove the rest of the package from the assembly.

- Attach the monitoring line to the male Luer connection on the transducer protector using aseptic techniques.
 - Perform dialysis using standard procedures.
CAUTION: If liquids (blood, dialysate, etc.) enter the unit and fill the housing, the Dualex™ Transducer Protector blocks the passage of air. If liquids fill the housing, clamp off the blood monitoring line and remove the transducer protector from the machine. Release the level of blood in the drip chamber, if necessary. Then attach a new Dualex™ Transducer Protector to the dialysis machine, and release the clamp.
5. Remove and dispose of the transducer protector after each dialysis.

Specifications

Materials

Membrane	Hydrophobic Durapel™ polyvinylidene fluoride (PVDF) membrane, type GV, pore size 0.2 µm
Housing	Polyvinyl chloride (PVC)
Dimensions	
Inlet to outlet	Dualex™ Transducer Protector: 25 mm (0.98 in.) Dualex™-Plus Transducer Protector: 25 mm (0.98 in.) Dualex™-Ultra Transducer Protector: 23 mm (0.91 in.)
Diameter	29 mm (1.14 in.)
Filteration area	4 cm² (0.62 in²)
Temperature limit	45 °C (113 °F)
Pressure limit at 25 °C	1.7 bar (25 psi) inlet and differential
Sterilization method	Ethylene oxide gas
Connections	Dualex™ Transducer Protector: Female Luer-slip inlet, male Luer-slip outlet Dualex™-Plus Transducer Protector: Female Luer-Lok™ inlet, male Luer-slip outlet Dualex™-Ultra Transducer Protector: Female Luer-Lok™ inlet, male Luer-Lok™ outlet

Notice

The information in this document is subject to change without notice and should not be construed as a commitment by Merck Millipore Ltd. („Millipore“) or an affiliate. Neither Merck Millipore Ltd. nor any of its affiliates assumes responsibility for any errors that may appear in this document.

Technical Assistance

For more information, contact the office nearest you. In the U.S., call 1-800-MILLIPORE (1-800-845-5476). Outside the U.S., go to our web site at www.millipore.com/offices for up-to-date world-wide contact information. You can also visit the tech service page on our web site at www.millipore.com/techservice.

Standard Product Warranty

The applicable warranty for the products listed in this publication may be found at www.millipore.com/terms (within the “Terms and Conditions of Sale” applicable to your purchase transaction).

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