

Hi-Flow™ Plus Membranes and SureWick® Pad Materials

Quality materials for use in rapid diagnostic lateral flow tests



Millipore®

Preparation, Separation, Filtration & Monitoring Products

The life science business of Merck KGaA, Darmstadt, Germany operates as MilliporeSigma in the U.S. and Canada.

# Quality Hi-Flow™ Plus Membranes sealed in an easy-to-use card format

As an industry leader and partner to global diagnostic manufacturers, we offer materials, components and custom services including membrane cards for rapid diagnostic test applications. The preassembled membrane cards reduce prep time and simplify your manufacturing design process and production. The precise assembly of our membrane cards results in exceptional consistency in performance for your test.

#### **Key Features of Hi-Flow™ Plus Membrane Cards**

#### Convenience

Buying membrane cards increases efficiency, providing you with a single source for high quality, lateral flow membranes on a plastic card support. The membrane cards also ensure optimal performance by eliminating the need to manually assemble cards for development or production of rapid diagnostic kits and tests.


## **Choose a Lateral Flow Membrane Card That Works to Your Advantage**

We offer laminated membrane cards in both standard and custom sizes, with five  $Hi\text{-}Flow^{\text{TM}}$  Plus membrane choices.

#### Hi-Flow™ Plus membrane card type has a distinctive flow range.

Speed	Sensitivity	Membrane Type	Description
Fast	Low	HF075, HF090	For assays where sample and analyte are abundant and sensitivity is not a major concern
Medium	Moderate	HF120, HF135	For assays that require moderate speed and sensitivity
Slow	High	HF180	For assays where sample is limited or high sensitivity is needed

#### Standard Hi-Flow™ Plus membrane card dimensions

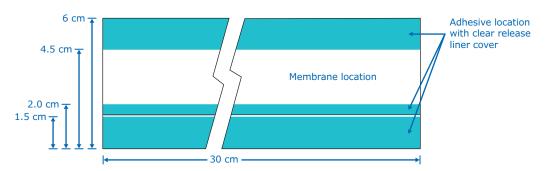


## Hi-Flow™ Plus Membranes

# For diagnostic tests and kits that require superior consistency and performance.

Hi-Flow™ Plus membranes are manufactured to the tightest specifications of any lateral flow membrane, and each membrane type has a characterized particle mobility for a given particle size. This membrane consistency results in reliable, lotto-lot performance of your rapid diagnostic test strip or device.

#### **Particle Mobility**

It is important to test mobility on the membranes which indicate a transition from blue to white, as these areas may show variability. The chart below illustrates which membranes allow for the greatest particle mobility.

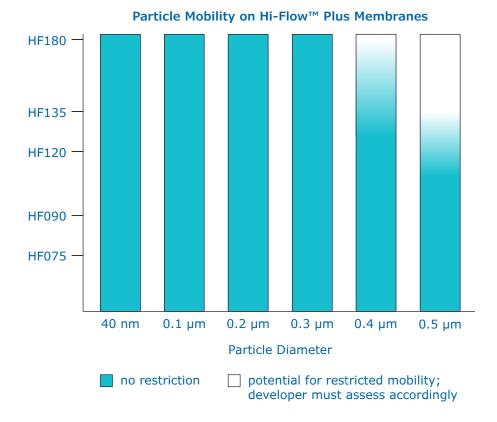

With colloidal gold there is little or no restriction on the flow of particles as they pass through the membrane. With larger beads, mobility may be restricted depending on the size of your particles and the viscosity of the sample.

In summary, it is important the assay developer understands the inherent mechanical structures throughout the lateral flow membrane which can pose physical barriers to particle flow. As a result, it is important to optimize particle/bead size with the desired speed and sensitivity of the chosen membrane.

#### **Key Features of Hi-Flow™ Plus Membranes**

#### **Consistent Flow and Thickness**

 $Hi ext{-}Flow^ op$  Plus membrane's consistent capillary flow time keeps the effective analyte concentration in your assay consistent. As a result, you can accurately determine the appropriate membrane for your application and develop assays with greater precision.



#### **Available Flow Rates\***

The capillary flow rate is the speed at which a sample material moves along a membrane strip in a rapid diagnostic test strip or application. Moreover, there is a direct correlation between the flow rate and sensitivity of the membrane. Understanding these requirements for your application will enable you to select the appropriate membrane. For example, you can use fast-flowing Hi-Flow™ Plus 75 for tests where speed is critical, sample is plentiful, and sensitivity is not a concern. On the other extreme, use slow-flowing Hi-Flow™ Plus 180 for tests where speed is less important, sample is limited, or high sensitivity is needed.

Use Hi-Flow  $\ensuremath{^{\text{TM}}}$  Plus membranes for assays that balance speed and sensitivity.


Membrane	Capillary Flow Time Specification** (sec/4 cm)	Flow Rate	Sensitivity
HF180	180 ± 45	Slowest	Most sensitive
HF135	135 ± 34	_ 	1
HF120	120 ± 30	_	
HF090	90 ± 23	- ♦	<b>★</b>
HF075	75 ± 19	- Fastest	Least sensitive


<sup>\*</sup> For additional details, visit the website to review the development guide for lateral flow strips: https://www.sigmaaldrich.com/technical-documents/articles

<sup>\*\*</sup> The range is for all measured values on a roll and represents  $\pm$  3 sec; the acceptable range for the mean is  $\pm$  10% of the target.

#### **Superior Surface Quality**

Hi-Flow™ Plus membranes are manufactured to have a superior surface morphology that allows for consistent and even flow distribution of reagents across the membrane surface. This excellent surface quality improves reagent addition and clarity of the test results.

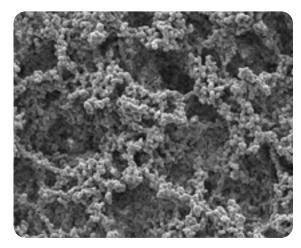

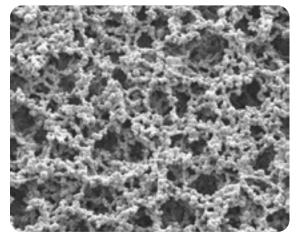

HF075 Air side view

HF180 Air side view

#### **Membrane Backing Options**

Membrane	Capillary Flow Time	Available Backing Material
HF075	75	2 mil and 4 mil
HF090	90	2 mil and 4 mil
HF120	120	2 mil and 4 mil
HF135	135	2 mil and 4 mil
HF180	180	UB*, 2 mil and 4 mil

<sup>\*</sup> unbacked

Also available are sample, absorbent and diagnostic pads needed for rapid diagnostic test strip applications and kits. These pad materials are standards in the industry and are available in multiple formats to suit your specific needs.

<sup>\*</sup>Only HF180 is available unbacked (UB).

# SureWick® Pad Material for use as Sample, Absorbent and Diagnostic Pads

We offer glass fiber diagnostic pads as well as nonwoven sample and absorbent pads. Both products are available in a variety of widths and formats (i.e. sheets or rolls) as a complement to a variety of manufacturing and development processes.


#### **Glass Fiber Diagnostic Pads**

Our glass fiber pads have low extractables and excellent consistency. These glass fiber pads are typically used as diagnostic pads in lateral flow test strip configurations. Advances in the production of these glass fiber substrates ensure uniform surfaces and low cost.


#### **Sample and Absorbent Pads**

Made from nonwoven, 100% pure cellulose fiber, these pads can be used as sample or absorbent pads. As with the diagnostic pads, these pads are manufactured to ensure consistency in your assay. These pads do not contain binders or glues that can interfere with assay performance.



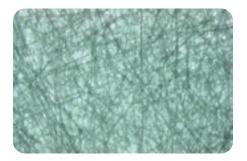

Glass Fiber Diagnostic Pad (6.3x)

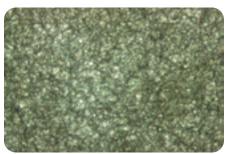

Cellulose Absorbent Sample Pad (6.3x)

# **Ordering Information**

#### **Hi-Flow™ Plus Membrane Cards**

Standard cards are available in the following specifications, however custom cards may also be ordered. Contact your local account manager to discuss the options.

#### **Membrane Cards**

Description	Card Dimension	Qty/Pack	Catalogue No.
Hi-Flow™ Plus 75 Membrane Cards	6 cm x 30 cm	100 cards	HF075MC100
Hi-Flow™ Plus 90 Membrane Cards	6 cm x 30 cm	100 cards	HF090MC100
Hi-Flow™ Plus 120 Membrane Cards	6 cm x 30 cm	100 cards	HF120MC100
Hi-Flow™ Plus 135 Membrane Cards	6 cm x 30 cm	100 cards	HF135MC100
Hi-Flow™ Plus 180 Membrane Cards	6 cm x 30 cm	100 cards	HF180MC100

#### **Sample Packs**

Description	Card Dimension	Qty/Pack	Catalogue No.
Hi-Flow™ Plus 75 Sample Pack	6 cm x 30 cm	5 cards	HF075MC5PK
Hi-Flow™ Plus 90 Sample Pack	6 cm x 30 cm	5 cards	HF090MC5PK
Hi-Flow™ Plus 120 Sample Pack	6 cm x 30 cm	5 cards	HF120MC5PK
Hi-Flow™ Plus 135 Sample Pack	6 cm x 30 cm	5 cards	HF135MC5PK
Hi-Flow™ Plus 180 Sample Pack	6 cm x 30 cm	5 cards	HF180MC5PK



#### **Hi-Flow™ Plus Membranes**

Standard membranes are available in the following formats, however custom slits and sheet sizes are also available. Contact your local account manager to discuss the options.

Variety Pack	Backing	Dimensions	Qty	Catalogue No.
Hi-Flow™ Plus 75, 90, 120, 135, 180	2 mil	25 cm x 30.5 cm sheets	10 sh/pk (2 sheets of each speed)	HFSAMPLE02
Hi-Flow™ Plus 75, 90, 120, 135, 180	4 mil	25 cm x 30.5 cm sheets	10 sh/pk (2 sheets of each speed)	HFSAMPLE04

#### Sheets: 25 cm x 30.5 cm

Description	Backing Thickness	Dimensions	Qty/Pack	Catalogue No.
Hi-Flow™ Plus 075	2 mil	25 cm x 30.5 cm sheets	5	HF07502XSS
Hi-Flow™ Plus 075	4 mil	25 cm x 30.5 cm sheets	5	HF07504XSS
Hi-Flow™ Plus 090	2 mil	25 cm x 30.5 cm sheets	5	HF09002XSS
Hi-Flow™ Plus 090	4 mil	25 cm x 30.5 cm sheets	5	HF09004XSS
Hi-Flow™ Plus 120	2 mil	25 cm x 30.5 cm sheets	5	HF12002XSS
Hi-Flow™ Plus 120	4 mil	25 cm x 30.5 cm sheets	5	HF12004XSS
Hi-Flow™ Plus 135	2 mil	25 cm x 30.5 cm sheets	5	HF13502XSS
Hi-Flow™ Plus 135	4 mil	25 cm x 30.5 cm sheets	5	HF13504XSS
Hi-Flow™ Plus 180	UB	25 cm x 30.5 cm sheets	5	HF180UBXSS
Hi-Flow™ Plus 180	2 mil	25 cm x 30.5 cm sheets	5	HF18002XSS
Hi-Flow™ Plus 180	4 mil	25 cm x 30.5 cm sheets	5	HF18004XSS

#### Rolls: 2.5 cm x 100 m

Description	Backing Thickness	Dimensions	Qty/Roll	Catalogue No.
Hi-Flow™ Plus 075	2 mil	2.5 cm W roll	100 m	SHF0750225
Hi-Flow™ Plus 075	4 mil	2.5 cm W roll	100 m	SHF0750425
Hi-Flow™ Plus 090	2 mil	2.5 cm W roll	100 m	SHF0900225
Hi-Flow™ Plus 090	4 mil	2.5 cm W roll	100 m	SHF0900425
Hi-Flow™ Plus 120	2 mil	2.5 cm W roll	100 m	SHF1200225
Hi-Flow™ Plus 120	4 mil	2.5 cm W roll	100 m	SHF1200425
Hi-Flow™ Plus 135	2 mil	2.5 cm W roll	100 m	SHF1350225
Hi-Flow™ Plus 135	4 mil	2.5 cm W roll	100 m	SHF1350425
Hi-Flow™ Plus 180	2 mil	2.5 cm W roll	100 m	SHF1800225
Hi-Flow™ Plus 180	4 mil	2.5 cm W roll	100 m	SHF1800425
Hi-Flow™ Plus 180	UB	2.5 cm W roll	100 m	SHF180UB25

#### Master Rolls: Slit to Order\*


Description	Backing	Length
Hi-Flow™ Plus 075	2 mil	100 m
Hi-Flow™ Plus 075	4 mil	100 m
Hi-Flow™ Plus 090	2 mil	100 m
Hi-Flow™ Plus 090	4 mil	100 m
Hi-Flow™ Plus 120	2 mil	100 m
Hi-Flow™ Plus 120	4 mil	100 m
Hi-Flow™ Plus 135	2 mil	100 m
Hi-Flow™ Plus 135	4 mil	100 m
Hi-Flow™ Plus 180	2 mil	100 m
Hi-Flow™ Plus 180	4 mil	100 m
Hi-Flow™ Plus 180	UB	100 m

 $<sup>^{\</sup>ast}$  A master roll is 30 cm wide x 100 meters long, and can be slit to a width that meets your design requirements. Contact your local account manager to obtain the catalog number.

A minimum order quantity is one full master roll (100 meters).


#### **Custom Products**

In addition to our standard products, we also have the ability to customize our membranes to improve assay performance in rapid diagnostic tests and applications. Contact your local account manager to discuss your assay optimization and product customization requirements.

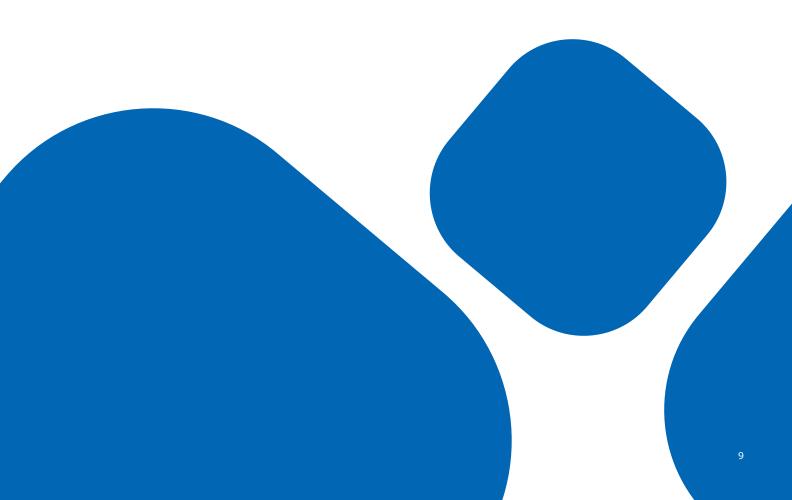

#### SureWick® Diagnostic Pads and Absorbent Pads — Standard Products

Description	Dimensions	Qty/Pk	Cat. No.		
GFDX Glass Fiber Diagnostic Pads					
Strips	10 mm x 300 mm	100	GFDX103000		
	8 mm x 300 mm	100	GFDX083000		
Sheets	20 cm x 30 cm	5	GFDX20305PK		
	20 cm x 30 cm	100	GFDX203000		
	20.32 cm x 25.4 cm	100	GFDX202500		
Rolls	8 mm x 50 m	1	GFDX000850		
	8 mm x 100 m	1	GFDX000800		
	10 mm x 50 m	1	GFDX001050		
	10 mm x 100 m	1	GFDX001000		
C083 Cellulose Absorbent S	C083 Cellulose Absorbent Sample Pads				
Strips	17 mm x 300 mm	100	CFSP173000		
	20 mm x 300 mm	100	CFSP203000		
Sheets	20 cm x 30 cm	5	CFSP22305PK		
	20 cm x 30 cm	100	CFSP223000		
Rolls	17 mm x 100 meters	1	CFSP001700		
	20 mm x 100 meters	1	CFSP002000		

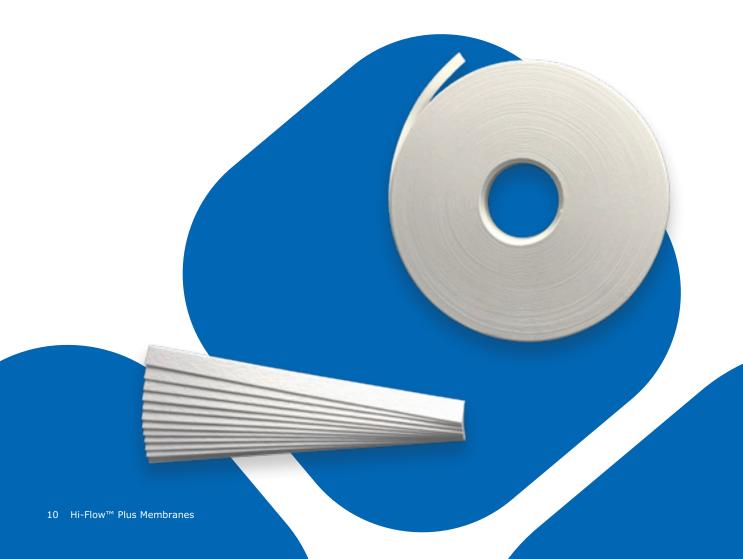

### **Quality Assurance You Can Trust**

All Hi-Flow™ Plus membrane materials are manufactured in our state-of-the-art manufacturing facility. We recommend storing these membranes and pad component materials in ambient humidity and temperature conditions. Store out of direct light while not in use during normal production.

## **Tighter Tolerances Reduce Your Incoming Quality Control (QC)**

Our tight release criteria ensure that your membrane will perform consistently between different product supply lots, which means minimal incoming QC testing on each new supply you receive.

#### **Detailed Certificate of Analysis**

Every lot of Hi-Flow  $^{\text{\tiny TM}}$  Plus membrane includes a certificate of analysis to provide details ensuring that:

- The lot complies with all test specifications
- There have been no changes in critical raw materials (membrane polymers)
- The membrane was produced in compliance with validated and documented manufacturing processes and material specifications

On each quality certificate, you will find the recommended storage temperature of 10-25  $^{\circ}$ C and 30-70 $^{\circ}$ RH as well as specific lot information and QC test data for your product.

#### **ISO, GMP Manufacturing**

Our manufacturing sites are ISO® 9001:2000 certified and GMP compliant. Customer plant audits are available upon request.

We provide the precision, compliant components and expert services you need to manufacture and optimize your diagnostic products. In the development and production of your lateral flow assay, we are your global partner from concept to clinic.

# consistency. Reliability.

# Millipore®

Preparation, Separation, Filtration & Monitoring Products

MilliporeSigma 400 Summit Drive Burlington, MA 01803

EMDMillipore.com

To place an order or receive technical assistance in the U.S. and Canada, call toll-free 1-800-645-5476 For other countries across Europe and the world, please visit: **EMDMillipore.com/offices** For Technical Service, please visit: **EMDMillipore.com/techservice**