User Guide

Millex®-MP Filter Unit (Sterile)

with Millipore Express® PES Membrane


REF	SLMP025SS*	SLMPL25SS
------------	-------------------	------------------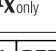


English	Español	Svenska	Česky	Türkçe
Français	Português	Latviski	Polski	Ελληνικά
Italiano	Nederlands	Lietuviškai	繁體中文	Slovenčina
Deutsch	Dansk	Magyar	Norsk	

The life science business of Merck KGaA, Darmstadt, Germany operates as MilliporeSigma in the U.S. and Canada.

PR06218	03/25	Date of issue:	2025-03
---------	-------	----------------	---------

Symbol	Definition	Symbol	Definition
	Sterilized using ethylene oxide		Use-by date
	Non-pyrogenic		Date of manufacture
	Consult instructions for use		Manufacturer
	Do not use if package is damaged and consult instructions for use		Caution
	Do not resterilize		CE conformity marking
	Do not re-use		Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare professional.
	Catalogue number		Swiss authorised representative
	Batch code		Unique device identifier
	Temperature limit		

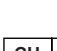	Made in Ireland
	Merck Millipore Ltd. Tullagreen, Carrigtwohill, Co. Cork, Ireland T45 KD29 an affiliate of Merck KGaA, Darmstadt, Germany
	MDSS CH GmbH Lauensteinvorstadt 61 5000 Aarau Switzerland
	Brazilian Registration Holder (BRH) Merck S/A Marante de Cajamar Business Center, South Access Road, Km 29,5-Km 29 plus 503 M, Warehouse Module B4 – Warehouse 01, Pólvora – Cajamar/SP/Brazil SP, Brazil ZIP CODE: 07790-330 * CNPJ no. 33.969.312/0008-50

Materials	Hydrophilic Millipore Express® polyethersulfone (PES), type MP
Pore size	0,22 µm
Housing	Polyvinyl chloride (PVC)
Dimensions	
Inlet to outlet	SLMP025SS: 25 mm (0.98 in.) <p>SLMPL25SS: 23 mm (0.91 in.)</p>
Diameter	29 mm (1.14 in.)
Filtration area	4 cm ² (0.62 in ²)
Temperature limit	45 °C (113 °F)
Pressure limit at 21 °C	5.2 bar (75 psi) inlet and differential
Filtration volume	1-100 mL
Hold-up volume	≤ 0,1 mL after air purge
Sterilization method	Ethylene oxide gas
Connections	SLMP025SS: Female Luer-Lok™ inlet; male Luer-slip outlet <p>SLMPL25SS: Female Luer-Lok™ inlet; male Luer-Lok™ outlet</p>
Flow rate at 2.1 bar (30 psi), 21 °C	≥ 175 mL/min

Notice
We provide information and advice to our customers on application technologies and regulatory matters to the best of our knowledge and ability, but without obligation or liability. Existing laws and regulations are to be observed in all cases by our customers. This also applies in respect to any rights of third parties. Our information and advice do not relieve our customers of their own responsibility for checking the suitability of our products for the envisaged purpose.

Contact Information
For the location of the office nearest you, go to [SigmaAldrich.com/offices](#).

Disposal
Follow precautions for disposal of items contaminated with potentially infectious or hazardous biological material according to all applicable international, federal, state, and local regulations.

Technical Assistance
Visit the tech service page on our web site at [SigmaAldrich.com/techservice](#). Any serious incident of this device should be reported to manufacturer and competent authority of country where user is established.

Standard Warranty
The applicable warranty for the products listed in this publication may be found at [SigmaAldrich.com/terms](#).

Made in Ireland

English
Milllex®-MP Filter Unit (Sterile)
with Millipore Express® PES Membrane
SLMP025SS Qty: 50 pack
SLMPL25SS Qty: 50 pack
• 25 mm • Sterile • Single use only • Non-pyrogenic

Intended Purpose
The Millex®-MP Filter Unit (Sterile) is medical device intended for use as a syringe filter to sterilize and/or clarify low volume solutions in direct patient care and pharmacy admixture applications. Device is intended for medical and/or laboratory professional use only.

Introduction
This document provides compatibility information, operating steps, and specifications for the Millex®-MP Filter Unit (Sterile). This filter is suitable for use in hospital pharmacy and direct patient care applications. In the hospital pharmacy, it can be used to sterile-filter the small volumes of protein pharmaceuticals, diagnostic imaging agents, chemotherapeutics, aqueous solutions, or water during admixture preparation. Direct patient care applications include sterilization and particulate removal from epidural and other liquid anaesthetics as well as from irrigation solutions used in ophthalmic, etc. and other surgical procedures.

This single-use product is non-pyrogenic and non-toxic. It consists of a modified polyethersulfone (PES) membrane filter sealed in a polyvinyl chloride (PVC) housing. Clinical benefits include the removal of microorganisms, particles, precipitates, and undissolved powders larger than the membrane's rated pore size from solutions for clinical use.


Chemical Compatibility
This filter is compatible with most aqueous solutions. Compatibility of solution should be tested before use. Visit [SigmaAldrich.com/MedMillex](#) for more information.

WARNINGS

- Filter solutions with a temperature limit of 45 °C (113 °F).
- Syringe filters are for manual use only; do not use on automated systems.
- Not suitable for filtering high viscosity solutions or blood.
- Do not filter solutions containing 5 mg or less of active drug materials unless binding studies have been performed.
- To ensure sterility, do not use this product if the package is damaged.
- Use aseptic technique to maintain sterility removing product from primary package.
- Do not use with syringes smaller than 10 mL because pressures in excess of the maximum pressure rating may be reached, potentially causing damage to the filter and/or personal injury.
- Do not use for direct neuraxial applications which may come in contact with cerebrospinal fluid.
- Do not use the same syringe filter to filter solutions in both directions.
- Sudden loss of pressure could indicate failure of the filter.
- Single use only; do not re-use or resterilize.
- Make sure to wet the filter membrane thoroughly before injecting the solution; improperly wetted filters can become airlocked.
- Not suitable for use with medical pressure infusion equipment or gravity fed intravenous (IV) administration sets.
- Discard appropriately after single use. See "Disposal" below.

Directions for Use

- Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare professional.
- Not suitable for use with medical pressure infusion equipment or gravity fed intravenous (IV) administration sets.
- Discard appropriately after single use. See "Disposal" below.

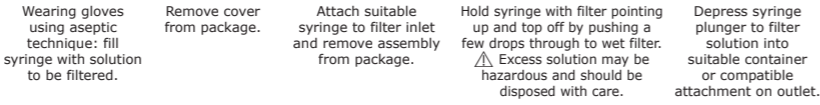Wearing gloves using aseptic technique: fill syringe with solution to be filtered.

Remove cover from package.

Attach suitable syringe to filter inlet and remove assembly from package.

Hold syringe with filter pointing up and top off by pushing a few drops through to wet filter. Excess solution may be hazardous and should be disposed with care.

Depress syringe plunger to filter solution into suitable container or compatible attachment on outlet.

Specifications and Performance Characteristics

Materials	Hydrophilic Millipore Express® polyethersulfone (PES), type MP
Pore size	0,22 µm
Housing	Polyvinyl chloride (PVC)
Dimensions	
Inlet to outlet	SLMP025SS: 25 mm (0.98 in.) <p>SLMPL25SS: 23 mm (0.91 in.)</p>
Diameter	29 mm (1.14 in.)
Filtration area	4 cm ² (0.62 in ²)
Temperature limit	45 °C (113 °F)
Pressure limit at 21 °C	5.2 bar (75 psi) inlet and differential
Filtration volume	1-100 mL
Hold-up volume	≤ 0,1 mL after air purge
Sterilization method	Ethylene oxide gas
Connections	SLMP025SS: Female Luer-Lok™ inlet; male Luer-slip outlet <p>SLMPL25SS: Female Luer-Lok™ inlet; male Luer-Lok™ outlet</p>
Flow rate at 2.1 bar (30 psi), 21 °C	≥ 175 mL/min

Notice
We provide information and advice to our customers on application technologies and regulatory matters to the best of our knowledge and ability, but without obligation or liability. Existing laws and regulations are to be observed in all cases by our customers. This also applies in respect to any rights of third parties. Our information and advice do not relieve our customers of their own responsibility for checking the suitability of our products for the envisaged purpose.

Contact Information
For the location of the office nearest you, go to [SigmaAldrich.com/offices](#).

Disposal
Follow precautions for disposal of items contaminated with potentially infectious or hazardous biological material according to all applicable international, federal, state, and local regulations.

Technical Assistance
Visit the tech service page on our web site at [SigmaAldrich.com/techservice](#). Any serious incident of this device should be reported to manufacturer and competent authority of country where user is established.

Standard Warranty
The applicable warranty for the products listed in this publication may be found at [SigmaAldrich.com/terms](#).

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