User Guide

MultiScreen[®]-HV and MultiScreen[®]_{HTS}-HV 96-well Plates

MAHVN4510
MAHVN4550
MSHVN4510
MSHVN4550

IVD For *in vitro* diagnostic use

Languages available

English	2	Lietuviškai	35
Français	5	Magyar	38
Italiano	8	Čeština	41
Deutsch	11	Polski	44
Español	14	日本語	47
Português	17	中文	50
Ελληνικά	20	한국어	53
Nederlands	23	Norsk	56
Dansk	26	Slovenčina	59
Svenska	29	Türkçe	62
Latviski	32		

Introduction

MultiScreen® filter plates containing 0.45 µm pore size membrane are intended for *in vitro* diagnostic use. They can be used to filter serum and other body fluids prior to IVD analysis. MultiScreen® 96-well plates are designed to filter samples and perform multiple procedures within the same plate. Materials supplied: Each non-sterile plate consists of a 96-well plate with integral filters, a plastic underdrain to prevent cross-contamination between wells, and a plastic top cover.

Classic MultiScreen®-HV plates should be used with a MultiScreen® Vacuum Manifold (Cat. No. MAVM0960R), or they can be centrifuged (Centrifuge Alignment Frame, Cat. No. MACF09604 required).

MultiScreen®_{HTS}-HV plates are automation-compatible for use with a variety of robotics systems and can be barcoded on all four sides. These plates should be used with a MultiScreen®_{HTS} Vacuum Manifold (Cat. No. MSVMHTS00), or they can be centrifuged using compatible accessories.

Device intended to be single-use and used by laboratory professional.

Usage Guidelines

- Do not remove the plastic underdrain from the plate before filtering samples. Once the underdrain has been removed, filtrate collection is not possible, even if the underdrain is subsequently re-attached.
- Do not place sealing tape over all the wells or leave the cover on the plate while applying vacuum, as this will prevent the flow of liquid through the filters.

How to Use MultiScreen® Plates

Some protocols require prewetting of the filter plate prior to use. If prewetting is not required for your application, continue on to the "Filtering Samples" section.

NOTE: After prewetting the plate, keep it damp before filtering, or it must be rewetted.

Aqueous Prewet for Filtration Assays

1. Place the plate on the appropriate vacuum manifold and remove the plate cover.
2. Add 100 µL buffer to each well. Vacuum after 1 minute. The plate is now ready for sample addition.

Alcohol Prewet for Filtration Assays

1. Place the plate on the appropriate manifold and remove the plate cover.
2. Add 50–100 µL of 70% ethanol to each well. After 30 seconds, filter by applying low vacuum.
3. To flush the residual ethanol from the wells, wash twice with 200 µL of starting buffer, using vacuum. The plate is now ready for sample addition.

Filtering Samples

The maximum recommended vacuum is 270–405 millibar (8–12 inHg). A higher vacuum level may be used for difficult-to-filter samples, but this may lead to higher filtrate controlled vacuum levels and sample foaming.

CAUTION: Do not use the manifold on the same bench or table as a vacuum pump, shaker, or mixer. The vibration may disrupt the filtrate transfer process for quantitative collection of filtrate.

1. Remove the plate cover and add solution(s) to the wells.
2. Replace the cover to minimize evaporation. Incubate per assay requirements.
3. Place the plate on the appropriate manifold.

CAUTION: Do not remove the plastic underdrain from the plate before filtering samples. Once the underdrain has been removed, filtrate collection is not possible, even if the underdrain is subsequently replaced.

4. Remove the cover and apply vacuum.

CAUTION: Empty wells will prevent flow. Add fluid to unused wells or cover unused wells with plate sealing tape.

5. Blot the plate on a lint-free absorbent surface to displace any microdroplets formed on the underside of the plate. Then add any other solutions that require further incubation.
6. Remove the plastic underdrain only if punching the samples out of the filtration plate with the Multiple Punch Assembly (Cat. No. MAMP09608).

CAUTION: To avoid contaminating the samples, do not touch the bottom of the plate.

Storage and Stability

Refer to product label for storage conditions and shelf life.

Chemical Compatibility

MultiScreen® plates are intended for use with biological fluids and aqueous solutions. Before use, check the sample for chemical compatibility with the device. Go to [SigmaAldrich.com/FilterChemicalCompatibility](https://www.sigmaaldrich.com/FilterChemicalCompatibility) for more information.

Specifications

	MultiScreen® Classic Plate	MultiScreen® Plate ^{HTS}	
Filter plate well capacity	300 µL	300 µL	
Working sample volume (capacity may be limited by receiver plate)	250 µL	250 µL	
Centrifugal speed (maximum)	1000 × g	3000 × g	
Overall Filter Plate Footprint	Length:	127.9 mm (5.0 in.)	127.8 mm (5.0 in.)
	Width:	85.7 mm (3.4 in.)	85.5 mm (3.4 in.)
	Height:	14.4 mm (0.6 in.)	14.4 mm (0.6 in.)
Materials of construction	Membrane:	0.45 µm hydrophilic Durapore® polyvinylidene fluoride	
	Cover:	Clear Styrene	
	Plate:	Clear Styrene	
	Underdrain:	Polyethylene	
Use with MultiScreen® Vacuum Manifold Cat. No.	MAVM0960R	MSVMHTS00	

Symbol Definitions

Symbol	Definition	Symbol	Definition
	<i>In vitro</i> diagnostic medical device		Do not use if package is damaged and consult instructions for use
	Catalogue number		Date of manufacture
	Batch code		Manufacturer
	Consult electronic instructions for use		Importer
	Download product documentation online		CE conformity marking
	Non-sterile		UK Conformity Assessed
	Do not reuse		Swiss authorized representative
	Use-by date		Unique device identifier
	Temperature limit		

Product Ordering

Purchase products online at SigmaAldrich.com.

Description	Qty/Pk	Catalogue Number
MultiScreen®-HV 96-well Plate	10 50	MAHVN4510 MAHVN4550
MultiScreen® _{HTS} -HV 96-well Plate	10 50	MSHVN4510 MSHVN4550

Notice

We provide information and advice to our customers on application technologies and regulatory matters to the best of our knowledge and ability, but without obligation or liability. Existing laws and regulations are to be observed in all cases by our customers. This also applies in respect to any rights of third parties. Our information and advice do not relieve our customers of their own responsibility for checking the suitability of our products for the envisaged purpose.

Collection and Disposal

All samples must be clearly labelled. Suitable instruments must be used for obtaining and preparing samples.

NOTE: Follow precautions for disposal of items contaminated with potentially infectious or hazardous biological material according to all applicable international, federal, state, and local regulations.

Contact Information

For the location of the office nearest you, go to SigmaAldrich.com/offices.

Technical Assistance

Visit the tech service page on our web site at SigmaAldrich.com/techservice.

Any serious incident of this device should be reported to manufacturer and competent authority of country where user is established.

Standard Warranty

The applicable warranty for the products listed in this publication may be found at SigmaAldrich.com/terms.

Revision History

2021-OCT	<ul style="list-style-type: none">• IFU PR05323 Date of Issue OCT 2021 - Replaced PR04138.• IFU and Packaging Damage symbols added.• Chemical Compatibility and Ordering Information linked to website.• Chemical Compatibility, Disposal and Complaints information added.• MultiScreen® plate dimensions aligned to Overall Plate Footprint• UK Responsible Person and UKCA symbol information added
2024-OCT	<ul style="list-style-type: none">• Added to Symbol Definitions table: CH-REP, Importer, UDI• Added UKCA symbol on title page

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Made in Ireland

Merck Millipore Ltd.
Tullagreen, Carrigtwohill,
Co. Cork, Ireland
T45 KD29

an affiliate of Merck KGaA,
Darmstadt, Germany

UK Responsible Person:

Millipore (UK) Ltd.
Kirkton Campus
Fleming Road,
Livingston, UK
EH54 7BN

Merck, Millipore, MultiScreen, Durapore, and Sigma-Aldrich are trademarks of Merck KGaA, Darmstadt, Germany or its affiliates. All other trademarks are the property of their respective owners. Detailed information on trademarks is available via publicly accessible resources.
© 1998-2025 Merck KGaA, Darmstadt, Germany and/or its affiliates. All Rights Reserved.

The life science business of Merck operates
as MilliporeSigma in the U.S. and Canada.