


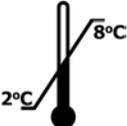

LIGHT DIAGNOSTICS™

 5093-5	HERPES SIMPLEX VIRUS (HSV) CONTROL SLIDES
 	

REPLACEMENT COMPONENT FOR:	<p>Light Diagnostics™ Herpes Simplex Virus (HSV) Control Slides are intended as reagent controls and may be used as a replacement component with:</p> <p>Light Diagnostics™ HSV 1/2 Typing DFA Kit  3291</p> <p>Light Diagnostics™ SimulFlour® HSV 1/2 DFA Kit  3293</p> <p>Light Diagnostics™ SimulFlour® HSV/VZV DFA Kit  3295</p>
FORMAT:	Each pack consists of five HSV antigen control slides containing one HSV type 1 infected (positive) well, one HSV type 2 infected (positive) well and one uninfected (negative) well. Slides are provided in individually sealed foil pouches containing desiccant.
APPLICATIONS:	Refer to Kit Product Insert for directions.
STORAGE/ HANDLING:	Store sealed slides at 2° to 8°C. Do not use slides past the expiration date printed on the label. Do not expose to elevated temperatures. Allow slides to come to room temperature prior to opening.
WARNINGS/ PRECAUTIONS:	<ol style="list-style-type: none">1. Handle all control slides and materials coming in contact with them as potentially infectious materials. Decontaminate appropriately.2. A control slide should be tested with each specimen batch to ensure proper function of the reagent and proper staining technique.

WARRANTY:

EMD Millipore warrants that its products will meet their applicable published specifications when used in accordance with their applicable instructions for the expressly stated duration. This warranty does not extend to any product which has been subjected to misuse, neglect, or to use in violation of instructions furnished by EMD Millipore. For products EXPRESSLY SOLD FOR USE IN FOOD, DRUG OR COSMETIC APPLICATIONS, EMD Millipore guarantees that no such product is adulterated or misbranded within the meaning of the Federal Food, Drug and Cosmetic Act (the "Act") or within the meaning of any other applicable law in which the definitions of adulteration or misbranding are substantially the same as those contained in the Act, as such laws are constituted and effective at the time of shipment, or is an article which may not, under Sections 404 or 505 of the Act, be introduced into interstate commerce.

EMD MILLIPORE MAKES NO OTHER WARRANTY, EXPRESS OR IMPLIED, AND EXPRESSLY DISCLAIMS WARRANTY OF MERCHANTABILITY OR WARRANTY OF FITNESS FOR A PARTICULAR PURPOSE. The warranty provided herein and the data, specifications and descriptions of EMD Millipore products appearing in EMD Millipore's published catalogues and product literature may not be altered except by express written agreement signed by an officer of EMD Millipore. Representations, oral or written, which are inconsistent with this warranty or such publications are not authorized and if given, should not be relied upon.

In the event of a breach of the foregoing product warranty, EMD Millipore's sole obligation shall be to repair or replace, at its option, the applicable product or part thereof, provided the customer notifies EMD Millipore promptly of any such breach. If after exercising reasonable efforts, EMD Millipore is unable to repair or replace the product or part, then EMD Millipore shall refund to the customer all monies paid for such applicable product or part.

EMD Millipore warrants that it shall provide services in a professional and workmanlike manner and that it will attempt in good faith to perform the services to Customer's reasonable satisfaction. **EMD MILLIPORE MAKES NO OTHER WARRANTY EXPRESS OR IMPLIED.** If EMD Millipore breaches this warranty, then Customer shall be entitled to a refund of up to the amount paid for such services. The above represents Customer's sole remedy for breach of this services warranty.

Buyer assumes all risk and liability for loss, damage or injury to persons or to property of Buyer or others arising out of the presence or use of the products or EMD Millipore's provision of services. Except as expressly provided otherwise herein, EMD Millipore shall not indemnify nor be liable to Buyer, Buyer's customers, successors, or to any person or entity for any claims, damages or losses arising out of the sale or use of products or the provision of services, where liability is premised upon any theory including, but not limited to, warranty, negligence or strict liability. **EMD MILLIPORE SHALL NOT IN ANY EVENT BE LIABLE FOR INCIDENTAL, CONSEQUENTIAL, INDIRECT, EXEMPLARY OR SPECIAL DAMAGES OF ANY KIND RESULTING FROM ANY USE OR FAILURE OF THE PRODUCTS OR SERVICES INCLUDING, WITHOUT LIMITATION, LIABILITY FOR LOSS OF USE, LOSS OF WORK IN PROGRESS, LOSS OF REVENUE OR PROFITS, OR ANY LIABILITY OF BUYER TO A THIRD PARTY. THE TOTAL LIABILITY OF EMD MILLIPORE UNDER THESE TERMS AND CONDITIONS OF SALE SHALL NOT EXCEED THE PURCHASE PRICE OF THE PRODUCTS OR SERVICES INVOLVED.** All claims must be brought within one (1) year of delivery, regardless of their nature.

©2012 EMD Millipore Corporation. a division of Merck KGaA, Darmstadt, Germany. All rights reserved. EMD, EMD Millipore, Millipore, the M mark, Light Diagnostics, SimuIFluor and all other trademarks, unless specifically identified above in the text as belonging to a third party, are owned by Merck KGaA, Darmstadt, Germany.



EMD Millipore Corporation
28820 Single Oak Drive Temecula, CA 92590 • United States
Tel. : +1 (951) 676-8080 • Fax : +1 (951) 676-9209
www.millipore.com



Millipore (UK) Ltd.
Fleming Road, Kirkton Campus • Livingston EH54 7BN
United Kingdom
Tel. +44 1506 404000 • Fax +44 1506 404001

Feb 2012
Revision A: 5093-5CA