User Guide

Millex®-25 0.2 µm Hydrophobic Vent Filter

SSLFGN25VS

Available languages:

English	2	Norsk (Norwegian)	22
Français (French)	4	Svenska (Swedish)	24
Italiano (Italian)	6	Latviski (Latvian)	26
Deutsch (German)	8	Lietuviškai (Lithuanian)	28
Español (Spanish)	10	Magyar (Hungarian)	30
Português (Portuguese)	12	Český (Czech)	32
Ελληνικά (Greek)	14	Polski (Polish)	34
Nederlands (Dutch)	16	繁體中文 (Traditional Chinese)	36
Dansk (Danish)	18	Русский (Russian)	37
Suomi (Finnish)	20		

Definition of Symbols Used

Symbol	Definition	Symbol	Definition
	Catalogue number		CE conformity marking
	Do not reuse		Date of manufacture
	Use-by date		Manufacturer
	Batch code		Do not use if package is damaged
			Sterilized using ethylene oxide

Millex®-25 0.2 µm Hydrophobic Vent Filter

- Single use only
- Sterile
- Non-pyrogenic

Indications for Use/Purpose

The bidirectionally supported Millex®-FG filter is a medical device intended for use as a syringe filter to sterilize and/or clarify low volume solutions or for air and gas in direct patient care and pharmacy admixture applications.

Introduction

For sterile venting of vials and bottles up to 2 liters. May also be used as a syringe filter. The bi-directionally supported Millex®-FG filter unit equipped with a needle is a sterilizing filter for gases and liquids that are not compatible with standard membrane filters. It will remove microorganisms, particles, precipitates, and undissolved powders larger than 0.2 micron (µm). The sterile Millex®-FG filter is non-pyrogenic and non-toxic. This single-use product consists of a 0.2 µm, hydrophobic Fluoropore™ membrane filter sealed in a polyvinyl chloride (PVC) housing. Typical applications include the sterile filtration of alcohols, concentrated acids and bases, and air or gas.

Directions for Use

WARNINGS

- **To ensure sterility, do not use this product if the package is visibly damaged.**
- **If used as a syringe filter, do not use with syringes smaller than 10 mL because pressures in excess of the maximum pressure rating may be reached, potentially causing damage to the filter unit and/or personal injury.**
- **Do not use as an in-line filter for intravenous administration; it was not designed for continuous long term use.**
- **Device not to be used for applications which may come in contact with cerebrospinal fluid.**

CAUTIONS

- Discard after single use. Do not resterilize or reuse this filter unit, as the company cannot assure the sterility, integrity, and performance beyond a single use.
- This product is not suitable for filtration of aqueous solutions.

Vent Filter Procedure

Use aseptic technique.

1. Spike the Millex®-25 vent into the vial of the drug (push the needle only partially through, so its opening is exposed in the vial).
2. Fill a syringe with diluent.
3. Add diluent to the vial.
4. Push the Millex®-25 vent filter into the vial up to the hub of the needle.
5. Hold the syringe, vial, and Millex®-25 vent filter, and agitate the vial to reconstitute the drug.
6. Invert the vial and draw the drug into the syringe.
7. Push any air out of the syringe, then remove the syringe from the vial.

Syringe Filter Procedure

Use aseptic technique.

1. Fill the syringe with the solution to be filtered.
2. Attach the syringe to the Millex®-25 filter.
3. Hold the syringe vertically with the filter pointing upward and top it off by pushing a few drops through the filter.
4. Place the filter assembly over the appropriate container and push the syringe plunger to deliver the filtered solution into the container.

Specifications

Materials

Membrane	Hydrophobic Fluoropore™ polytetrafluoroethylene (PTFE), type FG, pore size 0.2 µm
Housing	Polyvinyl chloride (PVC)
Needle	Stainless steel

Dimensions

Inlet to outlet including needle	61 mm (2.4 in.)
Diameter	28 mm (1.1 in.)
Filtration area	4 cm ² (0.62 in ²)

Temperature limit 45 °C (113 °F)

Maximum inlet pressure 2 bar (29 psi)

Sterilization method Ethylene oxide gas

Connections Female Luer-Lok™ inlet, Male Luer-slip outlet with needle

Notice

We provide information and advice to our customers on application technologies and regulatory matters to the best of our knowledge and ability, but without obligation or liability. Existing laws and regulations are to be observed in all cases by our customers. This also applies in respect to any rights of third parties. Our information and advice do not relieve our customers of their own responsibility for checking the suitability of our products for the envisaged purpose.

The information in this document is subject to change without notice and should not be construed as a commitment by the manufacturing or selling entity, or an affiliate. We assume no responsibility for any errors that may appear in this document.

Contact Information

For the location of the office nearest you, go to SigmaAldrich.com/offices.

Technical Assistance

Visit the tech service page on our web site at SigmaAldrich.com/techservice.

Standard Warranty

The applicable warranty for the products listed in this publication may be found at SigmaAldrich.com/terms.

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Made in Ireland
Merck Millipore Ltd.
Tullagreen, Carrigtwohill,
Co. Cork, IRL
an affiliate of Merck, KGaA, Darmstadt, Germany

Millipore, Millex, Fluoropore, and Sigma-Aldrich are trademarks of Merck KGaA, Darmstadt, Germany or its affiliates. All other trademarks are the property of their respective owners. Detailed information on trademarks is available via publicly accessible resources.
© 2021 Merck KGaA, Darmstadt, Germany and/or its affiliates. All Rights Reserved.

The life science business of Merck operates
as MilliporeSigma in the U.S. and Canada.