Millipore	SigmaAldrich.com
------------------	-------------------------

User Guide

Vented Millex®-AA Filter Unit (Sterile)

REF SLAAV255F

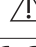
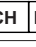
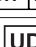
English	Português	Magyar
Français	Nederlands	Česky
Italiano	Dansk	Polski
Deutsch	Svenska	Norsk
Español	Latviski	日本語

CE ₀₄₅₉	R _{Xonly}
---------------------------	---------------------------

The life science business of Merck KGaA, Darmstadt, Germany operates as MilliporeSigma in the U.S. and Canada.

PRO6212	03/25	Date of issue: 2025-03
---------	-------	------------------------

Symbol	Definition	Symbol	Definition
STERILE	Sterilized using ethylene oxide		Use-by date
	Non-pyrogenic		Date of manufacture
	Consult instructions for use		Manufacturer
	Do not use if package is damaged and consult instructions for use		Caution
	Do not resterilize		CE conformity marking
	Do not re-use		Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare professional.
REF	Catalogue number	CH REP	Swiss authorised representative
LOT	Batch code	UDI	Unique device identifier
	Temperature limit		

	Made in Ireland		MOSS CH GmbH Laurenzvorstadt 61 5000 Aarau Switzerland
	Merck Millipore Ltd. Tullageen, Carrigtwohill, Co. Cork, Ireland T45 KD29 Darmstadt, Germany K83GA, Darmstadt, Germany		

The initial M, Millipore, Millex, MF-Millipore and Sigma Aldrich are trademarks of Merck KGaA, Darmstadt, Germany or its affiliates. All other trademarks are the property of their respective owners. Detailed information on trademarks is available via publicly accessible resources.

© 1991-2025 Merck KGaA, Darmstadt, Germany and/or its affiliates. All rights reserved.

	Deutsch	Español	Português	Nederlands
Belüftete Millex®-AA Filtereinheit (steril)	Unidad de filtración Millex®-AA (estéril) de venteo	Unidade de filtro Millex®-AA ventilada (estéril)	Unidade de filtro Millex®-AA ventilada (estéril)	Vented Millex®-AA Filtereenheid (Steriel)
SLAAV255F Menge: 50 St.Pkg.	SLAAV255F Cant.: 50 por envase	SLAAV255F Qtd.: 50/embalagem	SLAAV255F Qté.: 50/pack de 50	SLAAV255F Qtà: 50/conf.
• 25 mm • Steril • Nur für den Einmalgebrauch • Pyrogenfrei	• 25 mm • Estéril • Un solo uso • Apirógeno	• 25 mm • Estéril • Exclusivamente para uma única utilização • Não pirogênico	• 25 mm • Stérile • Strictement à usage unique • Apyrogène	• 25 mm • Steril • Uitsluitend voor eenmalig gebruik • Niet-pyrogeen

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Intended Purpose

The Vented Millex®-AA Filter Unit (Sterile) is a medical device intended for use as a syringe filter to clarify low volume solutions in direct patient care and pharmacy admixtue applications. Device is intended for medical and/or laboratory professional use only.

Introduction

This vented filter is a particulate filter for aqueous solutions dispensed with a syringe or dispensing unit. It will remove particles, precipitates, and undissolved powders larger than 0.8 µm. Clinical benefits include the removal of microorganisms, particles, precipitates, and undissolved powders larger than the membrane's rated pore size from solutions for clinical use. This sterile, single-use vented filter is non-pyrogenic and non-toxic. It consists of a 0.8 µm hydrophilic membrane filter sealed in a polyvinyl chloride (PVC) housing. A 0.2 µm hydrophobic filter in the dome prevents air locking by automatically venting air introduced upstream. Typical applications include the clarification of water, buffers, reconstituted antibiotics, ampouled drugs, bacterial toxins, antiseptic and allergenic preparations, and other aqueous solutions where protein binding is not a concern, as well as applications where air venting is required.

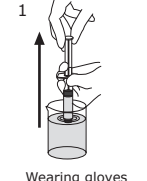
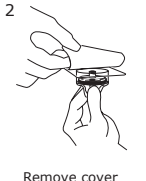
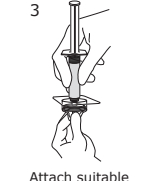
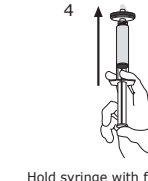
Chemical Compatibility

This vented filter is compatible with most aqueous solutions. Compatibility of solution should be tested before use. Visit [SigmaAldrich.com/MedMillex](#) for more information.

WARNINGS

- Filter solutions with a temperature limit of 45 °C (113 °F).
- Syringe filters are for manual use only; do not use in automated systems.
- Not suitable for filtering high viscosity solutions or blood.
- Do not filter solutions containing 5 mg or less of active drug materials unless binding studies have been performed.
- To ensure sterility, do not use this product if the package is damaged.
- Use aseptic technique to maintain sterility removing product from primary package.
- Do not use with syringes smaller than 10 mL because pressures in excess of the maximum pressure rating may be reached, potentially causing damage to the filter and/or personal injury.
- Do not use for direct neuralaxial applications which may come in contact with cerebrospinal fluid.
- This filter is not bi-directional. Do not use the syringe to filter in reverse direction.
- Sudden changes in pressure may damage filter.
- Sudden loss of pressure could indicate failure of the filter.
- Single use only; do not re-use or resterilize.
- Make sure to wet the AA filter membrane thoroughly before injecting the solution; improperly wetted filters can become airtight.
- Do not use this product as an in-line filter for gravity fed intravenous (IV) administration sets.
- Discard appropriately after single use. See "Disposal" below.

Directions for Use

				
Wearing gloves using aseptic technique: fill syringe with solution to be filtered.	Remove cover from package.	Attach suitable syringe to filter inlet and remove syringe from package.	Hold syringe with filter pointing up and top off by pushing a few drops through to wet filter. Excess solution may be hazardous and should be disposed with care.	Depress syringe plunger to filter solution into suitable container or compatible attachment on outlet.

Specifications and Performance Characteristics

Materials	
Filter membrane	Hydrophilic MF-Millipore™ mixed cellulose esters (MCE), type AA, pore size 0.8 µm
Vent membrane	Hydrophobic polytetrafluoroethylene (PTFE), pore size 0.2 µm
Housing	Polyvinyl chloride (PVC)

Dimensions	
Inlet to outlet	27 mm (1.06 in.)
Diameter	29 mm (1.14 in.)
Filtration area	4 cm² (0.62 in²)
Temperature limit	45 °C (113 °F)
Pressure limit at 21 °C	3.1 bar (45 psi) inlet and differential maximum
Filtration volume	1–100 mL
Hold-up volume	≤ 0.1 mL after air purge
Sterilization method	Ethylene oxide gas
Connections	Female Luer-Lok™ inlet; male Luer-slip outlet
Flow rate at 2.1 bar (30 psi), 21 °C	≥ 450 mL/min

Notice

We provide information and advice to our customers on application technologies and regulatory matters to the best of our knowledge and ability, but without obligation or liability. Existing laws and regulations are to be observed in all cases by our customers. This also applies in respect to any rights of third parties. Our information and advice do not relieve our customers of their own responsibility for checking the suitability of our products for the envisaged purpose.

Contact Information

For the location of the office nearest you, go to [SigmaAldrich.com/offices](#).

Disposal
Follow precautions for disposal of items contaminated with potentially infectious or hazardous biological material according to all applicable international, federal, state, and local regulations.

Technical Assistance

Visit the tech service page on our web site at [SigmaAldrich.com/techservice](#).

An serious incident of this device should be reported to manufacturer and competent authority of country where user is established.

Standard Warranty

The applicable warranty for the products listed in this publication may be found at [SigmaAldrich.com/terms](#).

	Deutsch	Español	Português	Nederlands
Belüftete Millex®-AA Filtereinheit (steril)	Unidad de filtración Millex®-AA (estéril) de venteo	Unidade de filtro Millex®-AA ventilada (estéril)	Unidade de filtro Millex®-AA ventilada (estéril)	Vented Millex®-AA Filtereenheid (Steriel)
SLAAV255F Menge: 50 St.Pkg.	SLAAV255F Cant.: 50 por envase	SLAAV255F Qtd.: 50/embalagem	SLAAV255F Qté.: 50/pack de 50	SLAAV255F Qtà: 50/conf.
• 25 mm • Steril • Nur für den Einmalgebrauch • Pyrogenfrei	• 25 mm • Estéril • Un solo uso • Apirógeno	• 25 mm • Estéril • Exclusivamente para uma única utilização • Não pirogênico	• 25 mm • Stérile • Strictement à usage unique • Apyrogène	• 25 mm • Steril • Uitsluitend voor eenmalig gebruik • Niet-pyrogeen

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