User Guide

Millex®-OR Filter Unit (Sterile)

with MF-Millipore™ Membrane

REF SLGL0250S

CE₀₄₅₉ Rx only

Available languages:

English	3	Latviski (Latvian)	21
Français (French)	5	Lietuviškai (Lithuanian)	23
Italiano (Italian)	7	Magyar (Hungarian)	25
Deutsch (German)	9	Česky (Czech)	27
Español (Spanish)	11	Polski (Polish)	29
Português (Portuguese)	13	Русский (Russian)	32
Dansk (Danish)	15	Norsk (Norwegian)	35
Nederlands (Dutch)	17	Türkçe (Turkish)	37
Svenska (Swedish)	19		

Definition of Symbols Used

Symbol	Definition	Symbol	Definition
	Sterilized using ethylene oxide		Use-by date
	Non-pyrogenic		Date of manufacture
	Consult instructions for use		Manufacturer
	Do not use if package is damaged and consult instructions for use		Caution
	Do not resterilize		CE conformity marking
	Do not re-use		Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare professional.
	Catalogue number		Swiss authorized representative
	Batch code		Unique device identifier
	Temperature limit		

Made in Ireland

Merck Millipore Ltd.
Tullagreen, Carrigtwohill,
Co. Cork, Ireland
T45 KD29

an affiliate of Merck KGaA,
Darmstadt, Germany

MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

Millex®-OR Filter Unit (Sterile) with MF-Millipore™ Membrane

SLGL0250S

Qty: 50 pack

- 25 mm
- Single use only
- Sterile
- Non-pyrogenic

Indications for Use/Purpose

The Millex®-OR Filter Unit (Sterile) is medical device intended for use as a syringe filter to sterilize and/or clarify low volume solutions in direct patient care and pharmacy admixture applications. Device is intended for medical and/or laboratory professional use only.

Introduction

This filter is a sterilizing filter for aqueous solutions dispensed with a syringe. Clinical benefits include the removal of microorganisms, particles, precipitates, and undissolved powders larger than the membrane's rated pore size from solutions for clinical use. This sterile, single-use filter is non-pyrogenic and non-toxic. It consists of a 0.22 µm hydrophilic mixed cellulose esters (MCE) membrane filter sealed in a polyvinyl chloride (PVC) housing. Typical applications include irrigation during ophthalmic surgery, preparation of epidural solutions, and other pharmacy admixtures.

Chemical Compatibility

This filter is compatible with most aqueous solutions. Compatibility of solution should be tested before use. Visit SigmaAldrich.com/MedMillex for more information.


WARNINGS

- Filter solutions with a temperature limit of 45 °C (113 °F).
- Syringe filters are for manual use only; do not use on automated systems.
- Not suitable for filtering high viscosity solutions or blood.
- Do not filter solutions containing 5 mg or less of active drug materials unless binding studies have been performed.
- To ensure sterility, do not use this product if the package is damaged.
- Use aseptic technique to maintain sterility removing product from primary package.
- Do not use with syringes smaller than 10 mL because pressures in excess of the maximum pressure rating may be reached, potentially causing damage to the filter and/or personal injury.
- Do not use for direct neuraxial applications which may come in contact with cerebrospinal fluid.

- Do not use the same syringe filter to filter solutions in both directions.
- Sudden loss of pressure could indicate failure of the filter.
- Single use only; do not re-use or resterilize.
- Make sure to wet the filter membrane thoroughly before injecting the solution; improperly wetted filters can become airlocked.
- Not suitable for use with medical pressure infusion equipment or gravity fed intravenous (IV) administration sets.
- Discard appropriately after single use. See "Disposal" section.

Directions for Use

1. Wearing gloves using aseptic technique: fill syringe with solution to be filtered.

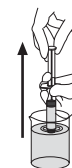2. Remove cover from package.


3. Attach suitable syringe to filter inlet and remove assembly from package.

4. Hold syringe with filter pointing up and top off by pushing a few drops through to wet filter.

⚠ Excess solution may be hazardous and should be disposed with care.

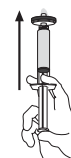5. Depress syringe plunger to filter solution into suitable container or compatible attachment on outlet.

Specifications and Performance Characteristics

Materials

Membrane	Mixed cellulose ester, type GS, pore size 0.22 µm
Housing	PVC (polyvinyl chloride)

Dimensions

Inlet to outlet	25 mm (0.98 in.)
Diameter	29 mm (1.14 in.)
Filtration area	4 cm ² (0.62 in ²)

Temperature limit	45 °C (113 °F)
--------------------------	----------------

Pressure limit at 21 °C	5.2 bar (75 psi) inlet and differential
--------------------------------	---

Filtration volume	1–100 mL
--------------------------	----------

Hold-up volume	≤ 0.1 mL after air purge
-----------------------	--------------------------

Sterilization method	Ethylene oxide gas
-----------------------------	--------------------

Connections	Female Luer-Lok™ inlet; male Luer-Lok™ outlet
--------------------	---

Flow rate at 2.1 bar (30 psi), 21 °C	≥ 75 mL/min
---	-------------

Notice

We provide information and advice to our customers on application technologies and regulatory matters to the best of our knowledge and ability, but without obligation or liability. Existing laws and regulations are to be observed in all cases by our customers. This also applies in respect to any rights of third parties. Our information and advice do not relieve our customers of their own responsibility for checking the suitability of our products for the envisaged purpose.

Contact Information

For the location of the office nearest you, go to SigmaAldrich.com/Offices.

Disposal

Follow precautions for disposal of items contaminated with potentially infectious or hazardous biological material according to all applicable international, federal, state, and local regulations.

Technical Assistance

Visit the tech service page on our web site at SigmaAldrich.com/TechService.

Any serious incident of this device should be reported to manufacturer and competent authority of country where user is established.

Terms and Conditions of Sale

Warranty, use restrictions, and other conditions of sale may be found at SigmaAldrich.com/Terms.

Made in Ireland

Millex®-OR Filter Unit (Sterile) with MF-Millipore™ Membrane

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Millipore, Millex, MF-Millipore and Sigma-Aldrich are trademarks of Merck KGaA, Darmstadt, Germany or its affiliates. All other trademarks are the property of their respective owners. Detailed information on trademarks is available via publicly accessible resources. © 2006-2023 Merck KGaA, Darmstadt, Germany and/or its affiliates. All Rights Reserved.

The life science business of Merck operates as MilliporeSigma in the U.S. and Canada.