User Guide

Millex®-OR/GS/HA Filter Unit (Non-Sterile)

REF SFGL02500 SLGS02510 SLHA02510

CE₀₄₅₉ Rx_{only}

Available languages:

English	3	Latviski (Latvian)	30
Français (French)	6	Lietuviškai (Lithuanian)	33
Italiano (Italian)	9	Magyar (Hungarian)	36
Deutsch (German)	12	Česky (Czech)	39
Español (Spanish)	15	Polski (Polish)	42
Português (Portuguese)	18	Русский (Russian)	45
Dansk (Danish)	21	Norsk (Norwegian)	48
Nederlands (Dutch)	24	Türkisch (Türkçe)	51
Svenska (Swedish)	27		

Definition of Symbols Used

Symbol	Definition	Symbol	Definition
	Non-sterile		Use-by date
	Non-pyrogenic		Date of manufacture
	Consult instructions for use		Manufacturer
	Do not use if package is damaged and consult instructions for use		Caution
	Do not re-use		CE conformity marking
	Catalogue number		Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare professional.
	Batch code		Swiss authorized representative
	Temperature limit		Unique device identifier

Made in Ireland

Merck Millipore Ltd.
Tullagreen, Carrigtwohill,
Co. Cork, Ireland
T45 KD29

an affiliate of Merck KGaA,
Darmstadt, Germany

MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

Millex®-OR/GS/HA Filter Unit (Non-Sterile)

Millex®-OR Filter	SFGL02500	Qty: 1000 pack
Millex®-GS Filter	SLGS02510	Qty: 1000 pack
Millex®-HA Filter	SLHA02510	Qty: 1000 pack

- 25 mm
- Non-sterile
- Single use only
- Non-pyrogenic

Intended Purpose

The Millex®-OR/GS/HA Filter Unit (Non-Sterile) are medical devices that are supplied in a non-sterile state. Once sterilized, they are intended for use as syringe filters to sterilize and/or clarify low volume solutions. Devices are intended for medical and/or laboratory professional use only.

Introduction

Millex®-OR/GS Filter Unit (Non-Sterile) are intended for clarifying small volumes of aqueous solutions, and once sterilized, can be used as a sterilizing filter for aqueous solutions. Millex®-HA Filter Unit (Non-Sterile) is intended for clarifying small volumes of aqueous solutions. Once repackaged and sterilized, Millex®-OR/GS/HA Filter Unit (Non-Sterile) can be used for direct patient care applications. The non-sterile filters are compatible with ethylene oxide (EO) sterilization. Sterilization by any other method (e.g., gamma irradiation, electron beam, steam) is not recommended.

Chemical Compatibility



These filters are compatible with most aqueous solutions. Compatibility of solution should be tested before use. Visit [SigmaAldrich.com/MedMillex](https://www.sigmaaldrich.com/MedMillex) for more information.

WARNINGS

- Sterilize with EO only.
- Filter solutions with a temperature limit of 45 °C (113 °F).
- Syringe filters are for manual use only; do not use on automated systems.
- Not suitable for filtering high viscosity solutions or blood.
- Do not filter solutions containing 5 mg or less of active drug materials unless binding studies have been performed.
- Do not use with syringes smaller than 10 mL because pressures in excess of the maximum pressure rating may be reached, potentially causing damage to the filter and/or personal injury.
- Do not use for direct neuraxial applications which may come in contact with cerebrospinal fluid.
- Do not use the same syringe filter to filter solutions in both directions.
- Sudden loss of pressure could indicate failure of the filter.
- Single use only; do not re-use.
- Make sure to wet the filter membrane thoroughly before injecting the solution; improperly wetted filters can become airlocked.
- Not suitable for use with medical pressure infusion equipment or gravity fed intravenous (IV) administration sets.
- Discard appropriately after single use. See "Disposal" below.

Directions for Use

1. Once sterilized, wear gloves and use aseptic technique. Fill syringe with solution to be filtered.
2. Attach suitable syringe to filter inlet and remove assembly from package.
3. Hold syringe with filter pointing up and top off by pushing a few drops through to wet filter.
⚠ Excess solution may be hazardous and should be disposed with care.
4. Depress syringe plunger to filter solution into suitable container or compatible attachment on outlet.

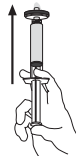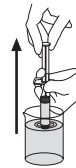Specifications and Performance Characteristics

Materials

Membrane	Hydrophilic MF-Millipore™ mixed cellulose esters (MCE) Millex®-OR filter: type GS, 0.22 µm
Pore size	Millex®-GS filter: type GS, 0.22 µm Millex®-HA filter: type HA, 0.45 µm
Housing	Polyvinyl chloride (PVC)

Dimensions

Inlet to outlet	Millex®-OR filter: 23 mm (0.91 in.) Millex®-GS filter: 25 mm (0.98 in.) Millex®-HA filter: 25 mm (0.98 in.)
Diameter	29 mm (1.14 in.)
Filtration area	4 cm ² (0.62 in ²)

Temperature limit

45 °C (113 °F) maximum

Pressure limit at 21 °C

5.2 bar (75 psi) inlet and differential

Filtration volume

1 mL to 100 mL

Hold-up volume

≤ 0.1 mL after air purge

Connections

Millex®-OR filter: Female Luer-Lok™ inlet, male Luer-Lok™ outlet

Millex®-GS filter: Female Luer-Lok™ inlet, male Luer-slip outlet

Millex®-HA filter: Female Luer-Lok™ inlet, male Luer-slip outlet

Flow rate at 2.1 bar (30 psi), 21 °C

Millex®-OR filter: ≥ 75 mL/min

Millex®-GS filter: ≥ 75 mL/min

Millex®-HA filter: ≥ 180 mL/min

Notice

We provide information and advice to our customers on application technologies and regulatory matters to the best of our knowledge and ability, but without obligation or liability. Existing laws and regulations are to be observed in all cases by our customers. This also applies in respect to any rights of third parties. Our information and advice do not relieve our customers of their own responsibility for checking the suitability of our products for the envisaged purpose.

Contact Information

For the location of the office nearest you, go to SigmaAldrich.com/Offices.

Disposal

Follow precautions for disposal of items contaminated with potentially infectious or hazardous biological material according to all applicable international, federal, state, and local regulations.

Technical Assistance

Visit the tech service page on our web site at SigmaAldrich.com/TechService.

Any serious incident of this device should be reported to manufacturer and competent authority of country where user is established.

Terms and Conditions of Sale

Warranty, use restrictions, and other conditions of sale may be found at SigmaAldrich.com/Terms.

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Merck, Millipore, Millex, MF-Millipore and Sigma-Aldrich are trademarks of Merck KGaA, Darmstadt, Germany or its affiliates. All other trademarks are the property of their respective owners. Detailed information on trademarks is available via publicly accessible resources.
© 2006-2023 Merck KGaA, Darmstadt, Germany and/or its affiliates. All Rights Reserved.

The life science business of Merck operates as MilliporeSigma in the U.S. and Canada.

MERCK