Millex® Vial Vent

SLGVS25LS

English
Français
Italiano
Deutsch
Español

Português
Ελληνικά
Nederlands
Dansk
Svenska

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Made in Ireland
Merck Millipore Ltd.
Tullagreen,
Carrigtwohill,
Co. Cork, IRL

© 2012 EMD Millipore Corporation. All rights reserved.
Millipore and Millex are registered trademarks of Merck KGaA, Darmstadt, Germany.
The M logo and Durapel are trademarks of Merck KGaA.
Luer-Lok is a trademark of Becton, Dickinson and Co.
PR04324, Rev. A, 08/12

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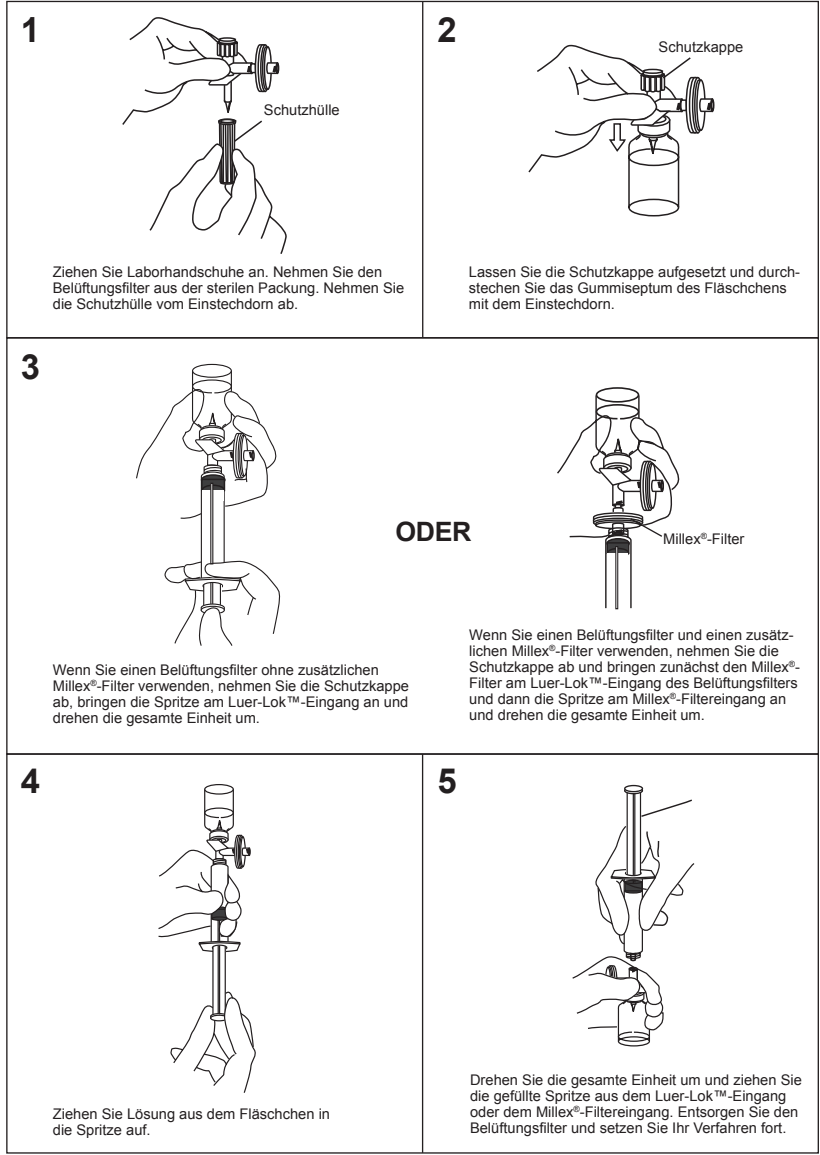
Millex® Vial Vent

Cat. No. SLGVS25LS (50/Box)

- Single use only
- Sterile
- Non-pyrogenic

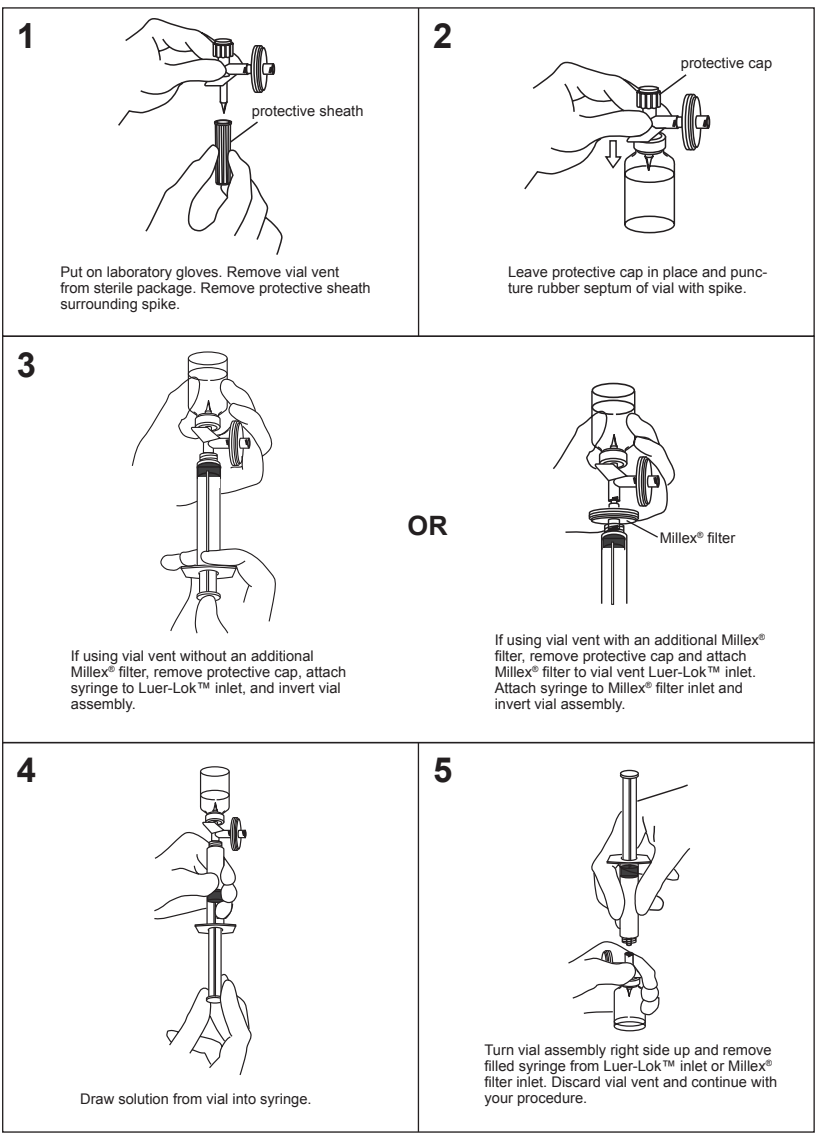
Applications

For use in pharmacy applications as a guard against inhaling hazardous compounds during reconstitution of drugs from sealed vials.

Directions for Use

WARNINGS/CAUTIONS:

- To ensure sterility, do not use this product if the package is damaged.
- Do not resterilize or reuse the Millex® Vial Vent, as Merck Millipore Ltd. cannot assure the sterility, integrity, and performance beyond a single use.
- Do not use the Millex® Vial Vent at temperatures above 45 °C (113 °F).

Specifications

Materials	
Vial vent membrane	Hydrophobic Durapel™ polyvinylidene fluoride (PVDF), pore size 0.22 µm
Millex® housing	Polyvinyl chloride (PVC)
Spike and cap	Acrylonitrile Butadiene Styrene (ABS)
Tip protector	Polyethylene
Dimensions	64 mm (2.5 in.) × 35 mm (1.4 in.) × 30 mm (1.2 in.)
Temperature limit	45 °C (113 °F)
Pressure limit at 25 °C	1.0 bar (15 psi) differential
Sterilization method	Ethylene oxide gas
Connections	Female Luer-Lok™ inlet, spike outlet

Notice

The information in this document is subject to change without notice and should not be construed as a commitment by Merck Millipore Ltd. („Millipore“) or an affiliate. Neither Merck Millipore Ltd. nor any of its affiliates assumes responsibility for any errors that may appear in this document.

Technical Assistance

For more information, contact the office nearest you. In the U.S., call 1-800-MILLIPORE (1-800-845-5476). Outside the U.S., go to our web site at www.millipore.com/offices for up-to-date world-wide contact information. You can also visit the tech service page on our web site at www.millipore.com/techservice.

Standard Product Warranty

The applicable warranty for the products listed in this publication may be found at www.millipore.com/terms (within the “Terms and Conditions of Sale” applicable to your purchase transaction).

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