

Fungi Agar Base acc. to KIMMIG, modified

Medium proposed by KIMMIG and RIETH (1953) for the cultivation, isolation, identification and strain preservation of fungi.



In Vitro Diagnostic Medical Device –

For professional use only



Version 13-01-2011

Merck KGaA, 64271 Darmstadt

*See also General Instruction for Use
„How to use Dehydrated Culture Media“*

*For MSDS, warnings and precautions see our website:
www.merck-chemicals.com*

Principle

Microbiological method.

General Information

This culture medium represents an improved version of the "Grütz II Agar" which is obtained by mixing it with MERCK Standard II Nutrient broth. According to RIETH (1969), it promotes the development of growth forms, which are used as important characteristic criteria for identification. KIMMIG agar can also be used as a base for preparing selective agars.

Principle

Microbiological method

Typical Composition (g/litre)

Peptone 15.0; sodium chloride 1.0; D(+)glucose 19.0; agar-agar 15.0.

Also to be added:

Glycerol 5.0.

Preparation

Suspend 50 g/litre together with 5 ml glycerol/litre, autoclave (15 min at 121 °C), pour plates.

pH: 6.5 ± 0.2 at 25 °C.

The plates are clear and yellowish-brown.

Preparation of selective agar:

Cool to approximately 50 °C, add 0.4 g cycloheximide/litre and, as recommended by GEORG et al. (1954), 40.000 IU penicillin/litre and 40 µg streptomycin/litre or, according to HANTSCHKE (1968), 80 mg colistin/litre and 100 mg novobiocin/litre and mix.

These compounds should be added in the form of filter-sterilized solutions. Pour plates.

Storage

Usable up to the expiry date when stored dry and tightly closed at +15 to +25° C. Protect from light.

After first opening of the bottle the content can be used up to the expiry date when stored dry and tightly closed at +15 to +25° C.

Specimen

e.g. Nails, hair, skin.

Clinical specimen collection, handling and processing, see general instructions of use.

Experimental Procedure and Evaluation

Inoculate the plates with the material, which should be obtained by an appropriate method. In the case of heavily contaminated material, use the selective agar described above or another one, for example Selective Agar for Pathogenic Fungi.

Incubation: up to 3 weeks at 25-28 °C. Identify the colonies. Identify the colonies.

Manufacturer	Product
Warner-Chilcott, USA	Colistin

Literature

GEORG, L.K., AJELLO, L., a. PAPAGEORGE, C.: Use of cycloheximide in the selective isolation of fungi pathogenic to man. - *J. Lab. Clin. Med.*, 44; 422-428 (1954). (1968).

HANTSCHKE, D.: Ein Colistin-Novobiocin-Actidion-Agar als Anzuchtmedium für humanpathogene Pilze. - *Mykosen*, 11; 769-778.

KIMMIG, J., u. RIETH, H.: Antimykotika in Experiment und Klinik. - *Arzneimittelforsch.*, 3; 267-276 (1953).

RIETH, H.: Dermatophyten, Hefen und Schimmelpilze auf Kimmig-Agar. - *Mykosen*, 12; 73-74 (1969).

Ordering Information

Product	Ordering No.	Pack size
Fungi Agar Base acc. to KIMMIG, modified	1.05414.0500	500 g
Merkoplate® Agar for fungi acc. to KIMMIG modified	1.10421.0001	1 x 20 plates
Glycerol (about 87 %)	1.04091.1000	1 l
Selective Agar for Pathogenic fungi	1.05467.0500	500 g
Novobiocin	491207	
Penicillin	516104	
	5161	
Streptomycin	51604	
	5711	

Fungi Agar Base acc. to KIMMIG, modified

Quality control

Test strains	Growth
<i>Microsporium gallinae</i> ATCC 12108	good / very good
<i>Trichophyton ajelloi</i> ATCC 28454	good / very good
<i>Trichophyton mentagrophytes</i> ATCC 18748	good / very good
<i>Microsporium canis</i> ATCC 36299	good / very good
<i>Penicillium</i> spp. ATCC 10428	good / very good
<i>Aspergillus brasiliensis</i> , formerly <i>A. niger</i> , ATCC 16404	good / very good
<i>Candida albicans</i> ATCC 10231	good / very good
<i>Geotrichum candidum</i> DSMZ 1240	good / very good