Instructions for Use

Weigert's Iron Hematoxylin Set

Procedure No. HT1079

Intended Use

Weigert's Iron Hematoxylin Set is commonly used as a histological nuclear stain in special staining techniques as it will not decolorize in acidic staining solutions.^{1,2} The data obtained from this manual qualitative procedure is used for the determination of chromatin in nuclei of human Histology specimens. This data can be used as an aid to diagnosis of certain clinical conditions or pathophysiological states and should be reviewed in conjunction with other clinical diagnostic tests or information.

Reagents

Weigert's Iron Hematoxylin Solution, Part A (Cat. No. HT107-500ML)
1% certified hematoxylin in ethanol

Weigert's Iron Hematoxylin Solution, Part B (Cat. No. HT109-500ML)

Ferric chloride 1.2% (w/v) and hydrochloric acid, 1% (v/v)

Special Materials Required but Not Provided

Positive control slides, such as Trichrome TISSUE-TROL™ should be included in each run.

Storage and Stability

Weigert's Iron Hematoxylin Set and Trichrome TISSUE-TROL™ should be stored at room temperature (18–26°C).

Reagents are stable until expiration date shown on the label.

Weigert's Iron Hematoxylin Working Solution should be prepared fresh for each use.

Preparation

Weigert's Iron Hematoxylin Working Solution is prepared by mixing equal parts of Weigert's Iron Hematoxylin Solution, Part A and Weigert's Iron Hematoxylin Solution, Part B.

Precautions

The IVDs included in these kits are intended for in vitro diagnostic use in a clinical laboratory environment. These IVDs are for professional use by qualified personnel only. Sigma-Aldrich IVDs may be operated by laboratory personnel who are trained to handle human specimens that can be infectious, use microscopes and other laboratory equipment and have color perception and visual acuity to distinguish colors and other objects under a microscope.

Normal precautions exercised in handling laboratory reagents should be followed. Dispose of waste observing all local, state, provincial or national regulations.

Trichrome TISSUE-TROL™ control slides are paraffin embedded human tissue containing collagen and muscle tissue and should be considered potentially infectious.

Procedure

Specimen Collection

No known test method can offer complete assurance that blood samples or tissue will not transmit infection. Therefore, all blood derivatives or tissue specimens should be considered potentially infectious.

Any well fixed paraffin section cut at 5 micron or less may be used.

Procedure

1. Prepare Weigert's Iron Hematoxylin Working Solution according to instructions.
2. Deparaffinize slides and hydrate to deionized water.
3. Stain in Weigert's Iron Hematoxylin Working Solution for **5 minutes**.
4. Wash in running tap water for **5 minutes**.
5. Rinse slides, dehydrate through alcohol, clear in xylene and mount.

Performance Characteristics

Target Structure	Staining Result
Nuclei	Blue to Black
Background	Unstained

If observed results vary from expected results, please contact Sigma-Aldrich Technical Service for assistance.

Analytical Performance Characteristics

The analytical performance results for the given tests conducted on all target structures, confirm 100% sensitivity, specificity, and repeatability.

Cat. No	Product Description	Target	Intra-assay Specificity	Intra-assay Sensitivity	Inter-assay Specificity	Inter-assay Sensitivity
HT107	Weigert's Iron Hematoxylin Solution, Part A	Nuclei	3 of 3	3 of 3	3 of 3	3 of 3
HT109	Weigert's Iron Hematoxylin Solution, Part B	Nuclei	3 of 3	3 of 3	3 of 3	3 of 3

Warnings and Hazards

Refer to Safety Data Sheet and product labeling for any updated risk, hazard or safety information.

HT1079:

H225: Highly flammable liquid and vapour.

H315: Causes skin irritation.

H319: Causes serious eye irritation.

P210: Keep away from heat/sparks/open flames/hot surfaces. No smoking.

P305 + P351 + P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

If during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Symbol Definitions

Symbols as defined in EN ISO 15223-1:2021

	Manufacturer		Catalogue Number
	Consult Instructions for Use		Batch Code
	Authorized Representative in the European Community/ European Union		European Union Declaration of Conformity (defined in IVDR 2017/746)
	Use-by Date		In vitro diagnostic medical device
	Temperature Limit		Caution
	Date of Manufacture		Importer
	Indicates the Authorised Representative in Switzerland		

References

1. Theory and Practice of Histotechnology, Edited by DC Sheehan and BB Hrapchak, 2nd ed, Mosby, St. Louis, (MO), 1980
2. Histotechnology: A Self Instructional Text. Carson, ASCP Press, Chicago, 1990

Contact Information

To place an order, please visit our web site at SigmaAldrich.com. For Technical Service, please visit the tech service page on our web site at SigmaAldrich.com/techservice.

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The Initial M, TISSUE-TROL, and Sigma-Aldrich are trademarks of Merck KGaA, Darmstadt, Germany or its affiliates. All other trademarks are the property of their respective owners. Detailed information on trademarks is available via publicly accessible resources.

© 2025 Merck KGaA, Darmstadt, Germany and/or its affiliates. All rights reserved.

The life science business of Merck operates as MilliporeSigma in the U.S. and Canada.

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Initial M, TISSUE-TROL и Sigma-Aldrich са търговски марки на Merck KGaA, Darmstadt, Германия или свързаните дружества на компанията. Всички други търговски марки са притежание на съответните им собственици. Подробна информация за търговските марки е налична чрез публично достъпни ресурси. © 2025 Merck KGaA, Darmstadt, Germany and/or its affiliates. All rights reserved.
Sigma-Aldrich, Inc.,
MDSS GmbH
MDSS CH GmbH
Chemie GmbH